

# ACCLEAN®

## CORDLESS HYGIENE HANDPIECE CLEANING & CARE GUIDE



Follow these steps after each use to properly clean, disinfect and prepare your Acclean Cordless Hygiene Handpiece for the next patient.

### STEP 1: DISASSEMBLE

After each use, take the unit apart in this order:

- Remove the disposable prophyl angle from the nosecone.
- Detach the nosecone from the handpiece and remove and discard the barrier sleeve.

### STEP 2: STERILIZE the NOSECONE *(do not use disinfectant solution)*

- Dampen a lint-free cloth with enzymatic detergent and thoroughly wet the nosecone.
- Use a soft-bristled brush to scrub until all visible soil is removed.
- Rinse under running water for at least 30 seconds.
- Dry with a clean, lint-free cloth and inspect for remaining soil. If necessary, repeat cleaning.
- Ensure nosecone is completely dry before sterilization.
- Place one nosecone per sterilization pouch.
- Sterilize using one of the following:
  - Gravity-displacement steam (ANSI/AAMI ST55): 132°C/269.6°F for 15 min + 30 min drying.
  - Dynamic-air removal steam (EN13060): 134°C/273.2°F for 4 min + 15 min drying.

### STEP 3: SURFACE CLEAN the HANDPIECE, BATTERY PACK, CHARGING BASE, and FOOT PEDAL

- Spray enzymatic cleaning solution onto a clean, lint-free cloth (not directly on components).
- Wipe the outer surfaces of the Handpiece, Battery Pack, Charging Base, Foot Pedal, and Power Cords (not pictured).
- Discard used cloth after cleaning.
- Allow all components to air dry before reuse or storage.



Nosecone



Prophy Angle



Battery Pack



Handpiece



Charging Base



Foot Pedal