


ACCLEAN®

CORDLESS HYGIENE HANDPIECE LIGHT INDICATOR GUIDE

CLEAN & POLISH

Easily navigate your Acclean® Cordless Hygiene HP with this simple visual guide to its six speed settings, battery light status and foot pedal pairing indicators.

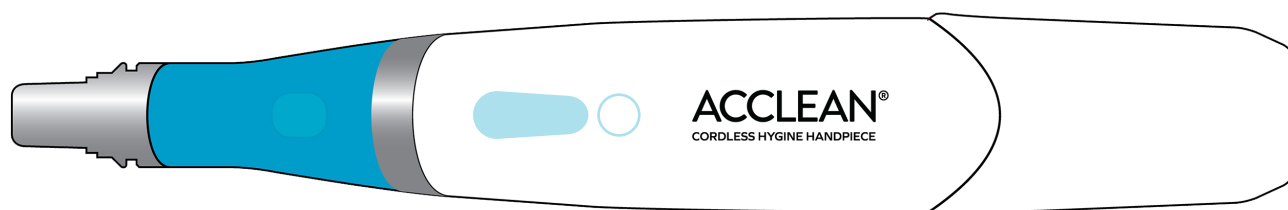

LED SETTINGS

| | | |
|----------|--------------------|--|
| 500 RPM | Off, Off, Flashing | |
| 1000 RPM | Off, Off, On | |
| 1500 RPM | Off, Flashing, On | |
| 2000 RPM | Off, On, On | |
| 2500 RPM | Flashing, On, On | |
| 3000 RPM | On, On, On | |

BATTERY LIFE

| | | |
|-------------|-------------|--|
| 25% to 0% | Solid Amber | |
| 100% to 25% | Solid Green | |

FOOT PEDAL PAIRING

| | | |
|--------------|---------------|--|
| Pairing Mode | Flashing Blue | |
| Connected | Solid Blue | |

SETTINGS

The last setting used before the unit is powered off will be stored and used the next time the unit is powered on.

AUTO POWER OFF

After five minutes of inactivity, the handpiece will shut off to save energy and prevent accidental operation.

CHARGING

The handpiece is ready to use if the battery life LED is green. If the LED is amber, remove the battery pack and replace it with a fully charged battery.

The first battery should be placed back in the base to be recharged, which takes approximately 65 minutes. A solid white ring on the base indicates the battery is fully charged.

Everyday Preventives for Healthy Patients

HenryScheinDental.com/Acclean

by  HENRY SCHEIN®



LEARN
MORE
AND
SHOP!